


Fanttest®



SARS-CoV-2/RSV/ Adenovirus/Influenza A+B Antigen Combo Rapid Test Kit

5-in-1 combination test kit,
detect SARS-CoV-2/RSV/Adenovirus/Influenza A+B
all at once.

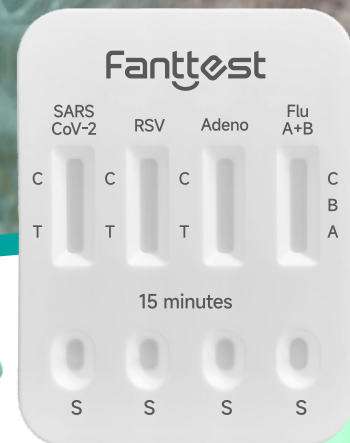

Methodology

Using the principle of double antibody sandwich method.



Operation

Simple operation, easy to interpret,
sampling and testing can be done
anytime.



Detection

The detection is fast, the result can be
interpreted in 15 minutes.



Accuracy

High sensitivity and specificity.



Hangzhou Fanttest Biotech Co., Ltd.


✉ sales@alfadiagnostik.com.my

☎ +0146217988

Intended use

SARS-CoV-2/RSV/Adenovirus/Influenza A+B Antigen Combo Rapid Test kit is a lateral flow immunoassay for the qualitative detection of SARS-CoV-2, respiratory syncytial, Adenovirus, influenza A and influenza B viral antigens in nasal/throat/nasopharyngeal swabs. The symptoms of respiratory viral infection due to SARS-CoV-2, respiratory syncytial, Adenovirus, influenza can be similar. The test is intended as an aid in diagnosis of symptomatic individual meeting the case definition for SARS-CoV-2 in the first 7 days of symptom onset and meeting the case definition for respiratory syncytial, Adenovirus, influenza A+B with in the first 4 days of symptom onset.

Operation steps and result interpretation

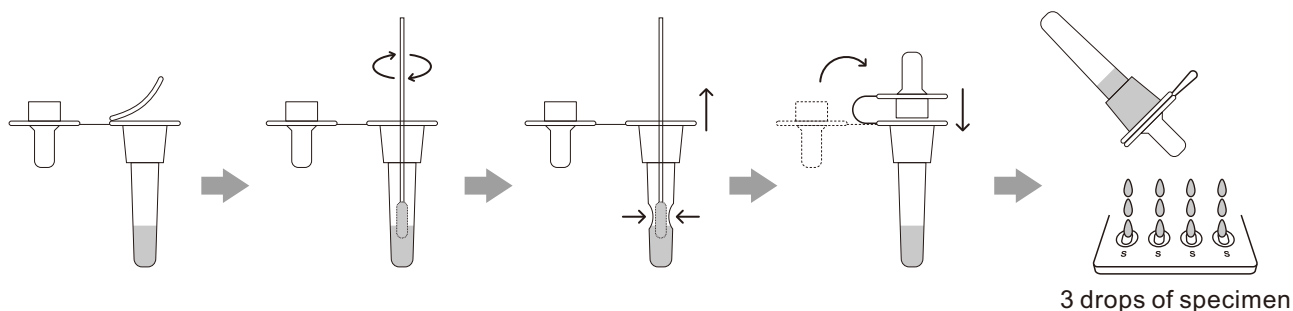

SARS-CoV-2/RSV/Adenovirus			Influenza A+B				
Positive	Negative	Invalid	Influenza B Positive	Influenza A Positive	Influenza A+B Positive	Negative	Invalid

SARS-CoV-2/RSV/Adenovirus

POSITIVE: Two colored lines appear on the membrane. One line appears in the control region (C) and the other line appears in the test region (T).

NEGATIVE: Only one colored line appears in the control region (C). No apparent colored line appears in the test region (T).

INVALID: Control line fails to appear.

Influenza A+B

Influenza A POSITIVE: It is positive for Influenza A antigen if two Red lines appear. One red line should be in the control line region (C), and the other one appears in the A test line region.

Influenza B POSITIVE: It is positive for Influenza B antigen if two Red lines appear. One red line should be in the control line region (C), and the other one appears in the B test line region.

Influenza A and B POSITIVE: It is positive for both the antigens of Influenza A and Influenza B if three red lines appear. One Red line should be in the control line region (C), and another two should appear in A test line region and B test line region.

NEGATIVE: One Red line appears in the control region (C). No apparent red line appears in the influenza A and B test region (T).

INVALID: Control line fails to appear.

Product information

Product name	Specimen	Format	Pack/box	Shelf life	Storage temperature	Certificate
SARS-CoV-2/RSV/Adenovirus/ Influenza A+B Antigen Combo Rapid Test Kit	Nasal swab/ Throat swab/ Nasopharyngeal swab	Cassette	1T/5T/25T	24months	2-30°C	CE